Resorcell Polvo

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Resorcell Powder

Sterile, resorbable, haemostatic powder
Hydrogencalcium salt of oxidised cellulose

Composition

100 % hydrogencalcium salt of oxidised cellulose.

Description

RESORCELL Powder is a sterile, resorbable, haemostatic powder designed for inner and external usage.

Characteristics

RESORCELL Powder is a sterile, resorbable, haemostatic powder, indicated to stop capillary bleeding. It acts immediately and provides fast and effective local haemostasia within 2 minutes after application depending on the type and the intensity of bleeding.

After the blood has saturated the colour of the powder changes to red till red-brown and changes to an adhesive mass that prevents the thrombocytes from being washed out of the wound thereby stopping bleeding faster. It minimises blood loss and complications perioperatively and in the early post-operative stages, eliminates the risk of blood transfer, has significant antibacterial activity, contributes to the acceleration of biochemical processes and facilitates the healing process. pH value of aqueous leach of product is 3,2 to 4,2. RESORCELL Powder is entirely biocompatible, atraumatic, fully biodegradable and is eliminated from the body without any adverse effects. RESORCELL Powder is well tolerated by the body and does not cause any undesirable immune reactions or sensitization when used repeatedly.

Indications

RESORCELL Powder is indicated to stop capillary bleeding and to prevent bleeding in the early dental post-operative stages. The product is indicated for use by adults and children.

Contraindications

RESORCELL Powder is not designed to be used to stop arterial bleeding.

Undesirable effects

Undesirable side effects not known. In case of unusual reaction, contact your doctor.

Interactions

No interaction has not been reported yet.

Method of application

After opening and fixating the disperser by repeatedly compressing the walls of the bottle, apply a small layer of the haemostatic powder to the bleeding area and allow it to dry. It is not necessary to remove the excess powder from the wound. RESORCELL Powder is atraumatic and fully biodegradable.

Warning

Although RESORCELL Powder has antibacterial effect, it is not intended as a substitute for systemically administered therapeutic or prophylactic antimicrobial agents. RESORCELL Powder is not designed to be used to stop arterial bleeding. Washing off the RESORCELL Powder of the wound can be taken as a wrong method of application.

The product is made for a single usage only.

Product sterility is ensured only by undamaged coverage.

The product cannot be resterilised.
RESORCELL Powder must not be used after the expiry date stated on the package.

Storage

Do not expose to the direct light. Store in original packaging, in a dry place at a temperature 2 – 30°C.

Presentation

2 g plastic bottle of powder with a plastic disperser.
They are both packaged in a peelable blister, sterilised by gamma rays.

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