Tray-net

POLVO Y RECIPIENTE

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Composición

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Tray-net

POWDER AND TANK
Cleaning product for impression trays

Composition

Sodium citrate, cetrimide, excipients.

Properties

TRAY-NET disintegrates alginates, or plaster on impression trays by simple immersion followed by rinsing in water, without scrubbing.

TRAY-NET can be used on all plastic or metallic impression trays (**except aluminium trays**) and also to clean spatulas and alginate mixing bowls.

TRAY-NET is safe for hands.

Directions for use

- Remove the metal accessories from the tank.
- Using the measuring spoon, put three measures of powder into the tank. If you do not have the TRAY-NET tank, pour into your container used for that purpose 5 measures of powder per liter of water, using the measuring spoon.
- Half fill the tank with hot water.
- Replace the metallic support and add water to the metal flap level.
- Hang the impression trays on the support rod, then dip them into the TRAY-NET solution.
- Let stand into the solution for a few hours or, preferably, overnight and then rinse the impression trays under running water (only "nickel silver" trays and plastic trays used with an adhesive require light scrubbing).

Notes

- The tank will hold 5 or 6 impression trays at the same time.
 - For maximum efficiency the solution should be changed after each use ; it is, however, possible to keep the prepared solution for the treatment of 15 to 20 trays, in groups of 5 or 6.
- TRAY-NET has no effect on silicone or other elastomeric impression materials.

Presentation

- Box containing 1 kg of TRAY-NET powder with measuring spoon.
- Cleaning tank and accessories.

For professional use only in the practice of dentistry.

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