(FR)

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(EN)

The Detartrine range of products are ready-to-use dental abrasive pastes for cleaning and polishing.

**Detartrine** and **Detartrine Z** are pink pastes with a nice taste.

Their mechanical abrasive action is due to the presence of silica (in Detartrine) and of silica and zirconium (in Detartrine Z). It does not cause erosion of dental enamel.

**PRESENTATION**

45g or 150g jar

**INTENDED USE**

Dental abrasive paste for cleaning and polishing

**TARGET GROUP(S)**

**Target population**

Children and adults with permanent teeth

**Intended users**

Dental healthcare professional

**CLINICAL BENEFIT**

Prevention of caries and periodontal diseases (prophylaxis)

Prevention of soft tissue injuries

Aesthetic enhancement

**PERFORMANCE CHARACTERISTICS**

**Technical performance / specifications**

Relative dentin abrasivity (RDA) :

- Detartrine : 143.63 ± 23.86
- Detartrine Z : 156.13 ± 24.89

**Clinical performance**

Removal of dental plaque
Reduction of extrinsic stains
Smoothing of tooth and restorative material surfaces

**INDICATIONS FOR USE**

Polishing of scaled surfaces
Polishing of restorative materials
Removal of dental plaque (bacterial biofilm)
Reduction of food and tobacco stains

**CONTRA INDICATION**

Hypersensitivity to one of the ingredients
Deminerlized teeth

**INSTRUCTIONS FOR USE**

Take a small amount of paste with a spatula and place it on a brush or any other suitable finishing instrument.
Mouth rinsing can be performed during or at the end of the scaling procedure.

**WARNINGS AND PRECAUTIONS FOR USE**

**Precautions for use**

Keep the product in its original packaging.
Close the jar immediately after use.

**SIDE-EFFECTS**

No known side effects.

Please report any serious incident that occurred in relation to the device to Septodont (vigilance@septodont.com) and to the local competent authority for medical devices.

**STORAGE**

Store between 5 and 30 °C.
Keep in the upright position.

**DISPOSAL**

After use of the jar and/or any excessive amount of paste, place in an appropriate container.

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