EN SeptoPlus Mandrel / SeptoDiscs Mandrel

Accessory device for fixing different finishing and/or polishing systems in contra-angle. Available in two systems: mandrel by pressure (SeptoPlus Mandrel) and pop-on mandrel (SeptoDiscs Mandrel).

Use: Install the mandrel at the contra-angle. Fit the disc onto the mandrel. SeptoPlus Mandrel: Compatible with SeptoPolisher and SeptoWheels PX. SeptoDiscs Mandrel: Compatible with SeptoDiscs.

Warnings, precautions and restrictions: Professional use only. The user will be entirely responsible for the mandrel or the disc; with similar products. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. If there is proven allergy to any of its components, the product must not be used.

Cleaning and Sterilization: The mandrel is autoclavable (126 ° C, 147 kPa, 16 min). Before sterilization, clean using water and enzymatic detergent. The durability of the mandrel is directly related to care during use.

Composition: Metal.
Storage: To be stored in a cool and dry place, protected from sunlight.
Connection with contra-angle: The SeptoPlus Mandrel is manufactured in compliance with specifications of ISO 1797. The SeptoDiscs Mandrel is manufactured in compliance with specifications of ISO 1797.

Presentación:
0110010004 - SeptoPlus Mandrel - 1 unit
0110000005 - SeptoDiscs Mandrel - 1 unit

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TD Dental Ltd.
SEPTODONT, Inc