FR

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EN

Composition
Acetone, Ethyl acetate, Phenylethyl alcohol, Amyl acetate.

Intended use

Dehydrating and degreasing agent.
Clinical benefit
Facilitate dental preparation leading to functional tooth.

Intended population
Children and adults with mature permanent teeth (from eight years old).

Indications for use
Hydrol is a dehydrating and degreasing agent for:

• dental cavities before filling
• dental preparation before restoration procedure (prosthetic pieces)

Contraindications

Hypersensitivity to one of components

Use on vital teeth

Performance

Dehydrate and degrease restorative material and dental cavities during endodontic treatment.

Instructions for use

Dental cavities before filling:

The operative field shall be completely isolated with a rubber dam around the tooth.

1. Put some product in an appropriate container.
2. Soak a cotton ball in the product.

3. Softly dab the cavities to be dried and degreased.
4. Wait until complete evaporation of the liquid. Make sure there is no product left in the cavity.

5. Close the bottle after use.
6. Any remaining product that was sampled from the bottle shall be discarded.

Dental preparation before restoration procedure

(prosthetic pieces):

1. Put some product in an appropriate container.
2. Soak a cotton ball in the product.

3. Dab the prosthetic piece to be dried and degreased.
4. Wait until complete evaporation of the liquid. Make sure there is no product left on the prosthetic piece.

5. Close the bottle after use.
6. Any remaining product that was sampled from the bottle shall be discarded.

Warnings and precautions for use**Precautions for use**

Handle the bottle carefully.

Keep the product in its original packaging.
Do not apply excessive amount of Hydrol.

Warnings
Do not use Hydrol on the gingival mucosa, as prolonged application may cause an irritation.

Highly flammable liquid and vapour.
May cause drowsiness or dizziness.

Causes serious eye irritation.
Side effects

Hypersensitivity reaction

Disposal

After use, dispose the product in an appropriate container.

Storage

Do not store above 25 °C.

Presentations

Bottle containing 45 mL of solution

Bottle containing 250 mL of solution

For dental healthcare professional**ES**

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